


# Iskio b-m Cushion

**Expert advice from Sonia Baudry Occupational Therapist**

*I recommend the Iskio b-m cushion for people with a tendency to slide forward or to the side. A correct sitting position improves functional stability and allows the patient to perform everyday activities more easily.*



Delivered with 1 3D mesh cover and 1 Pharmatex cover

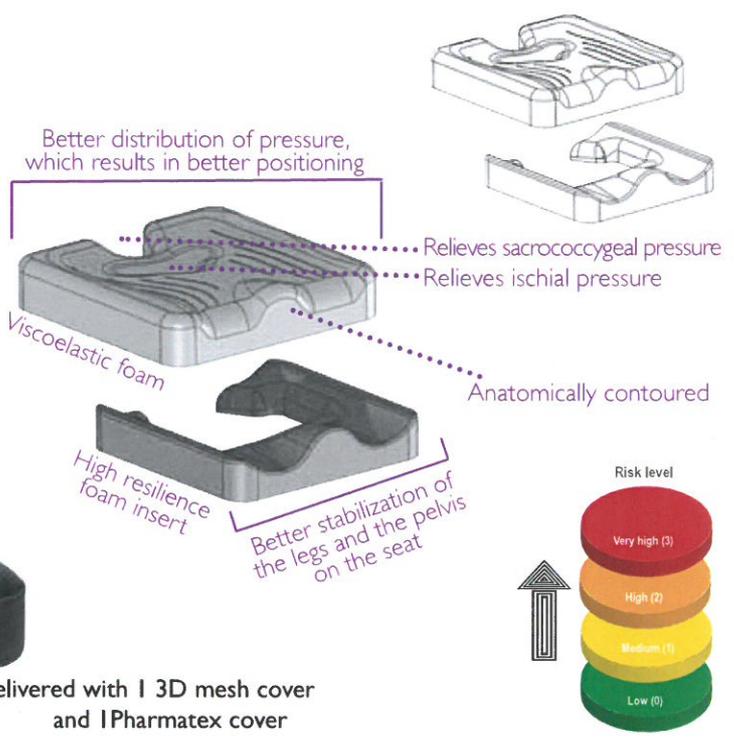

The ISKIO b-m cushion is made of viscoelastic foam with a high resilience foam insert. Thanks to its numerous beneficial features, this technical cushion is able to provide a better distribution of pressure. The high resilience foam insert increases the weight-bearing capacity for certain areas in order to ensure that pressure is relieved from bony prominences in at-risk areas. This is done by distributing pressure to lower risk areas such as the thighs, lesser trochanters and along the periphery of the buttocks.

- The crural incline prevents retroversion of the pelvis (pelvis sliding forwards)
- The rear cutout and seat shape are specifically designed to relieve pressure in the ischial and sacrococcygeal areas while preventing the cushion from "bottoming out", thus increasing the comfort of the seated person. This cushion, particularly due to its 10.5 cm height, is suited to persons who remain seated in chairs for long periods of time.

## Recommendations for use

Use of the cushion is designed for people with a risk of pressure ulcers most often linked to static posture problems while sitting (sliding to the front of the seat, with too much support on the sacrococcygeal area), for use either at home or in care facilities for dependent elderly persons.

## Materials

- Viscoelastic foam: 80 kg/m<sup>3</sup>
- H.R foam insert: 37 kg/m<sup>3</sup>
- 3D mesh cover: polyester, + film polyurethane
- Pharmatex cover: 75 % polyurethane, 25 % polyester
- Anti-slip bottom: 55 % polyurethane, 45 % polyester
- Fire standards: EN597-1 and EN597-2

## References and dimensions

Reference	Width		Depth		Height	
	mm	inch	mm	inch	mm	inch
<b>17.17106</b>	400	16"	400	16"	85	3 1/4"
<b>17.17108</b>	400	16"	400	16"	105	4 1/4"
<b>17.17116</b>	430	17"	430	17"	85	3 1/4"
<b>17.17118</b>	430	17"	430	17"	105	4 1/4"
<b>17.17126</b>	460	18"	430	17"	85	3 1/4"
<b>17.17128</b>	460	18"	430	17"	105	4 1/4"
<b>17.17136</b>	500	19 1/2"	450	17 3/4"	85	3 1/4"
<b>17.17138</b>	500	19 1/2"	450	17 3/4"	105	4 1/4"

## Care tips

Cushion cleaning: same as the Iskio Visco  
 Cover cleaning: same as the Iskio Visco